

The NAPCO 9000D
Operational Manual
Brought To You Compliments of
www.AllClaveparts.com

Please Note: This is NOT a free e-book!

Purchase of this e-book entitles the buyer to keep one copy on his or her computer and to print out one copy only.

Printing out more than one copy – or distributing it electronically – is prohibited by international and U.S.A. copyright laws and treaties, and would subject the purchaser to penalties of up to \$100,000 PER COPY distributed


All Rights Reserved

Copyright © MMIX AllClaveParts. All rights are reserved. You may not distribute this report in any way. You may not sell it, or reprint any part of it without written consent from the author, except for the inclusion of brief quotations in a review.

AllClaveParts


National Appliance 704-9000 Operational Manual



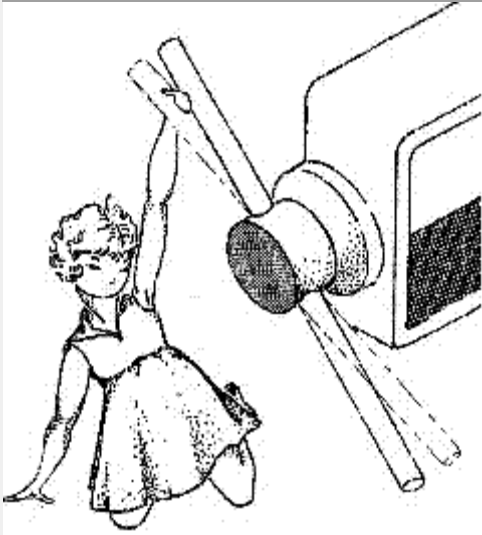

Step 1


Open The Door By Turning The Handle Counter-Clockwise And Swing Yoke Off The Door Bar Remove Contents From Chamber

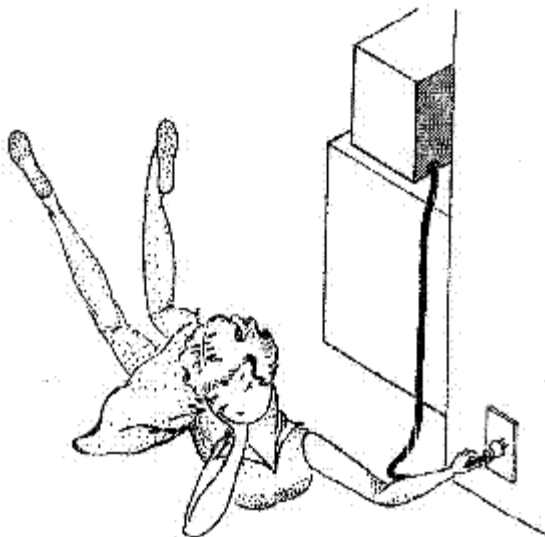

Make Sure Power & Timer
Are Plugged Into Electrical Outlet

Step 3

See Water (H₂O) Level Indicator On The Bottom Of The Shelf In Autoclave Chamber Add Water (H₂O)

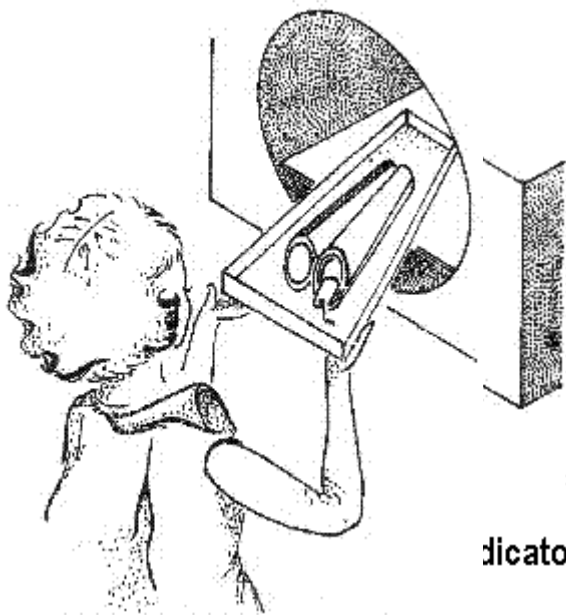

Step 4


Load The Autoclave Tray. Pack Loosely To Obtain Evenly Distributed Flow Of Steam Around And Through The Packs

Step 5

It Is Recommended That A Steam Clox


icator

Sterilization Indicator Be Used With Each Load. Insert Sterilization Indicator Into Pack And Slide Tray Onto Shelf In Autoclave Chamber

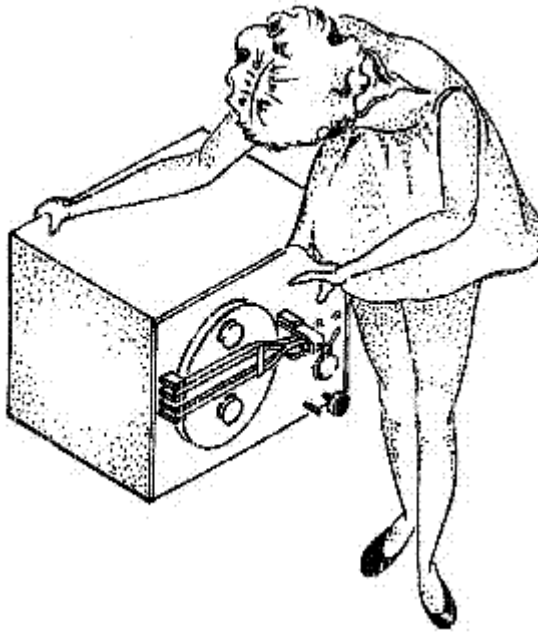

Step 6



Close Door And Tighten Securely

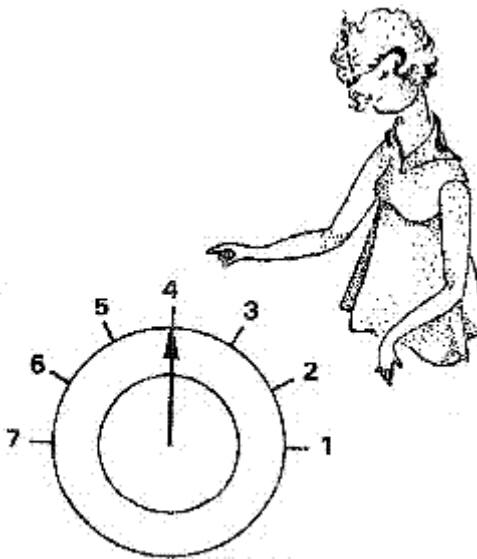

Step 7


Set Temperature Control To Center Of Dial Position (Position 4). This Position Will Set Thermometer Pointer In The Red Area On The Face Of The Thermometer At About 127°C (260°F)

Turn Switch To Autoclave Only
Turn Power On

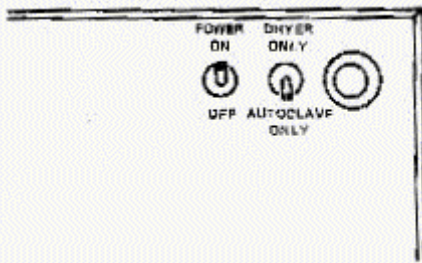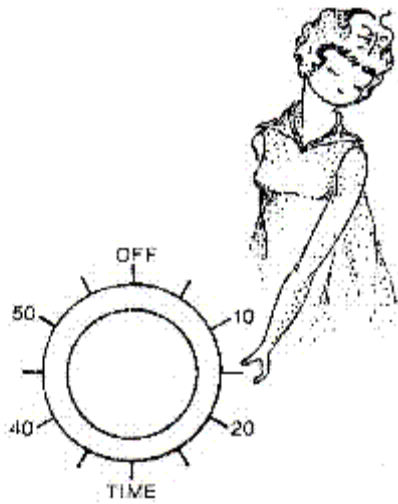


Step 8

From A Cold Start It Takes About 15
Minutes To Warm Up The Chamber To
Autoclaving Temperature Set Timer For
10 Minutes

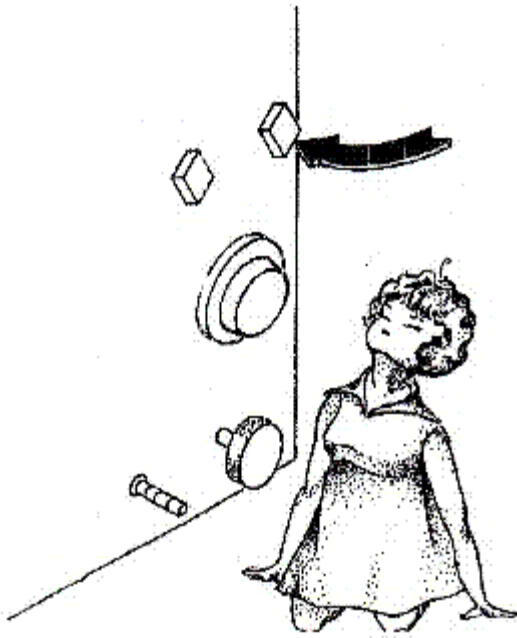

Step 9

When Temperature Light Goes Out, The Chamber Has Reached The Pre-Set
Temperaure

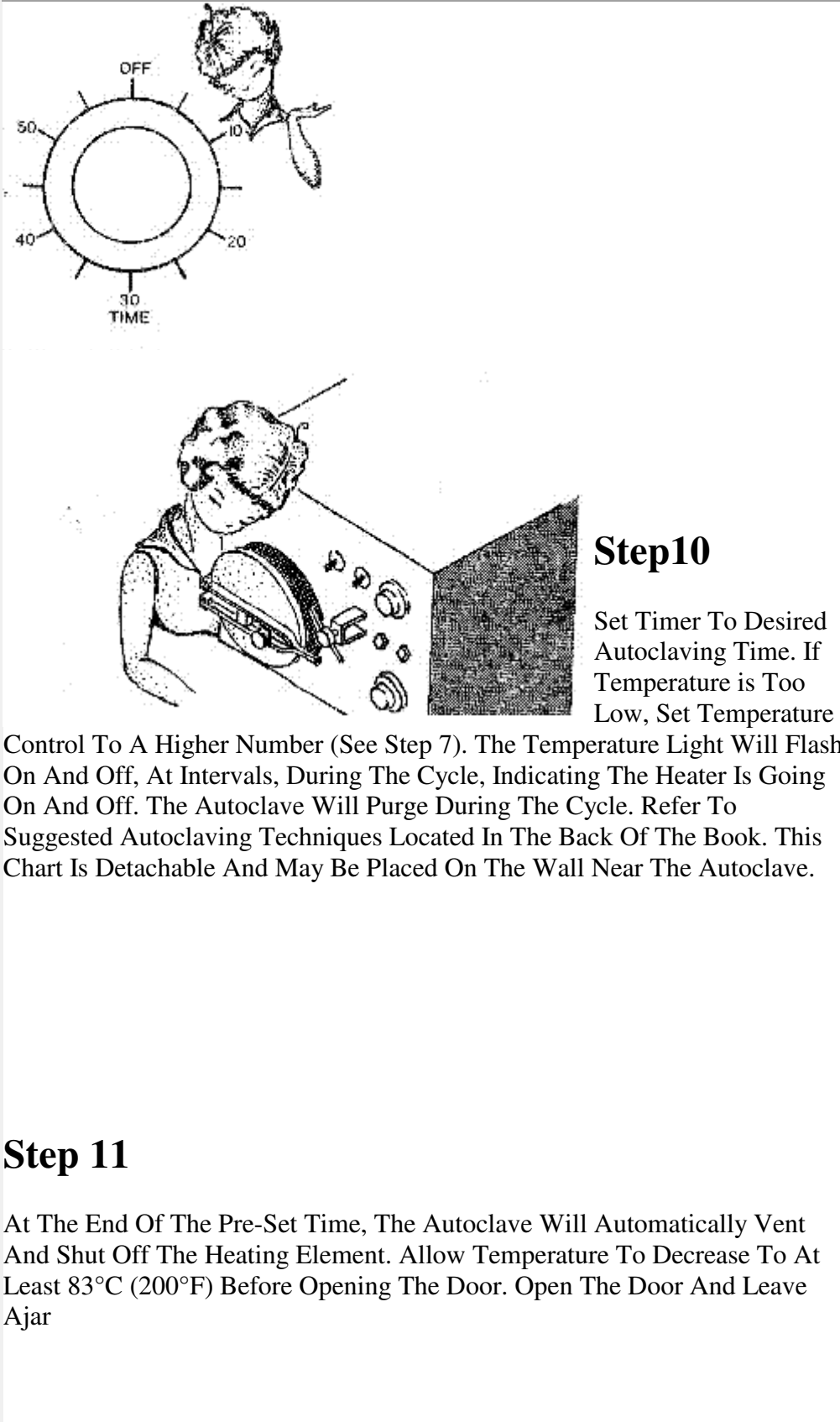


Step 10

Set Timer To Desired Autoclaving Time. If Temperature is Too Low, Set Temperature

Control To A Higher Number (See Step 7). The Temperature Light Will Flash On And Off, At Intervals, During The Cycle, Indicating The Heater Is Going On And Off. The Autoclave Will Purge During The Cycle. Refer To Suggested Autoclaving Techniques Located In The Back Of The Book. This Chart Is Detachable And May Be Placed On The Wall Near The Autoclave.

Step 11

At The End Of The Pre-Set Time, The Autoclave Will Automatically Vent And Shut Off The Heating Element. Allow Temperature To Decrease To At Least 83°C (200°F) Before Opening The Door. Open The Door And Leave Ajar

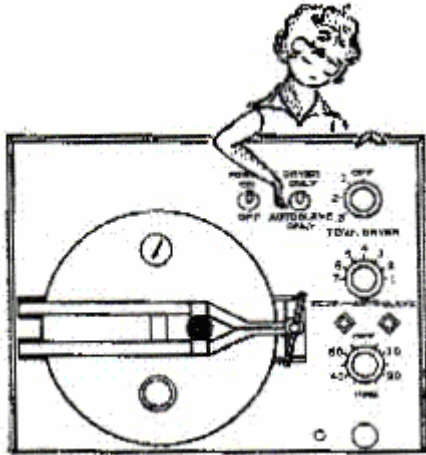


Step 12

1. To Operate Drying Cycle Turn Toggle Switch To Dryer Only. Make Sure Door Is Slightly Ajar Throughout Drying Cycle
2. Turn Temperature Dryer Control To Position 1
3. Set Timer For 15 Minutes. The Time Light Will Come On And

The Temperature Light Will Stay Off

4. When Timer Turns Off Check Contents And Remove If Dry. Use A Higher Drying Temperature If Desired But Use Caution To Avoid Scorching Contents

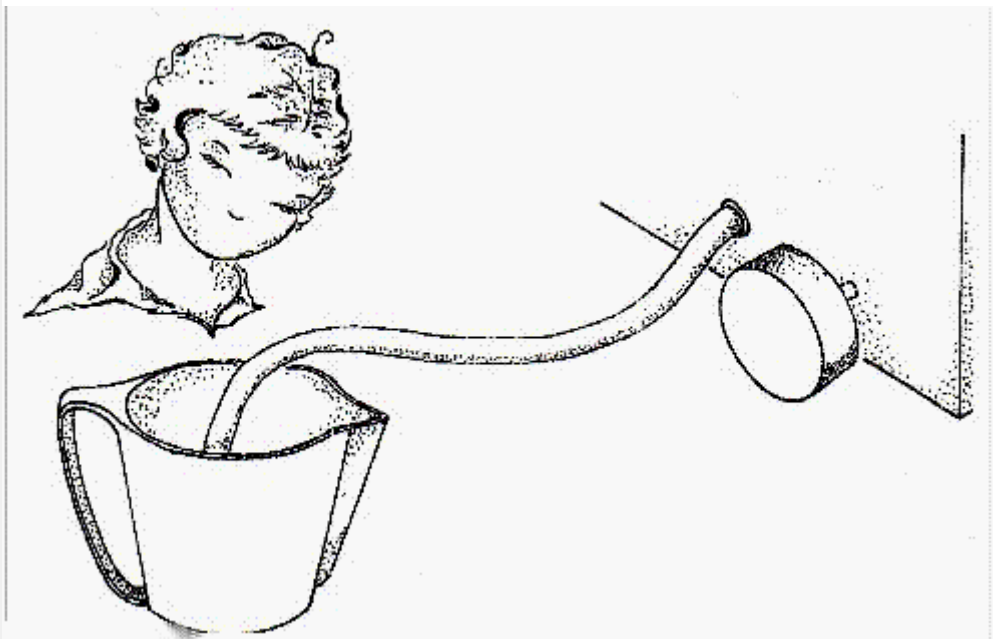

Step 13



Before The Next Cycle, Drain The Condensor Tank. This Tank Should Be Drained After Each Cycle And The Chamber Should Be Filled As Illustrated In Step 3

Step 14

Star Next Cycle By Repeating Steps 1 Through 7

Be Sure To Check The Water (H2O) Level Before Setting The Timer



Step 15

After The First Run, The Unit Is Warmed Up And For Subsequent Runs You Can Set The Timer To Combine Warm-Up And Autoclave Time. The Unit Will Reach Sterilization Temperature In Approximately 10 Minutes